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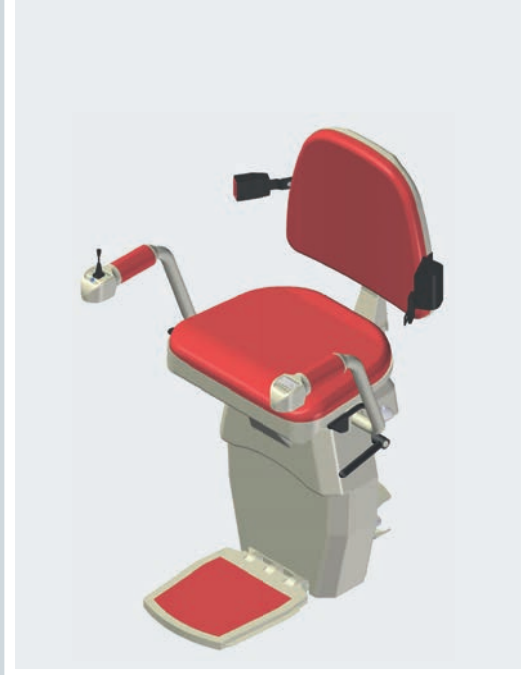
KURVENLIFT MODELL L – 06 (HW10/HW11)

D

INSTRUCTIONS FOR USE

STAIRLIFT MODEL L – 06 (HW10/HW11)

GB

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LIFT DE TREPTÉ MODEL L – 06 (HW10/HW11)

RO

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Name:
Anschrift:
Telefon:
E-Mail:

1. SUPPLIER DETAILS	pag. 20
2. PURCHASE INFORMATION FOR BUYERS	pag. 20
3. FEATURES	pag. 20
3.1 GENERAL DESCRIPTION	
3.2 TECHNICAL DATA	
3.3 ENVIRONMENTAL INFLUENCE	
4. USE	pag. 24
4.1 ELEMENTS OF USE	
4.2 PREVENTIVE MEASURES BEFORE USE	
4.3 PROHIBITIONS	
5. MAINTENANCE	pag. 33
5.1 BY THE USER	
5.2 BY AUTHORIZED COMPANIES	
5.3 MAINTENANCE LIST	

1. SUPPLIER DETAILS

If you have any questions regarding the operation or maintenance of your stair lift, please contact your supplier by phone or in writing.

Supplier name:
Address:
Phone no.:
E-mail:

2. PURCHASE INFORMATION FOR BUYERS

If maintenance or repairs under guarantee are needed, your supplier will need the following information:

Client name:
Address:
Phone no.:
Date of purchase:
Stair lift serial no.:

3. FEATURES

3.1 GENERAL DESCRIPTION

The L - 06 (HW10/HW11) stair lift was designed to help adults with locomotor disabilities or the senior citizen have access to on two or even three levels inside their home or building. The elevator can be installed both on the inside, and on the outside of the staircase, to the left or right.. To occupy as little space in the parking area, it can be folded. To call the stair lift from the parking area you will use the remote control. For your safety, this lift type was equipped with all necessary guards and some features (e.g., electrical rotation of the seat).

The L-06 stair lift travels on a track consisting of two parallel pipes with different diameters arranged one above the other for optimal retrieval of loads, the two pipes being rigidly interconnected by means of mountings that connect them with the steps via some foot supports fastened with screws to the track mountings and fastened to the steps with screws – fig. 1.

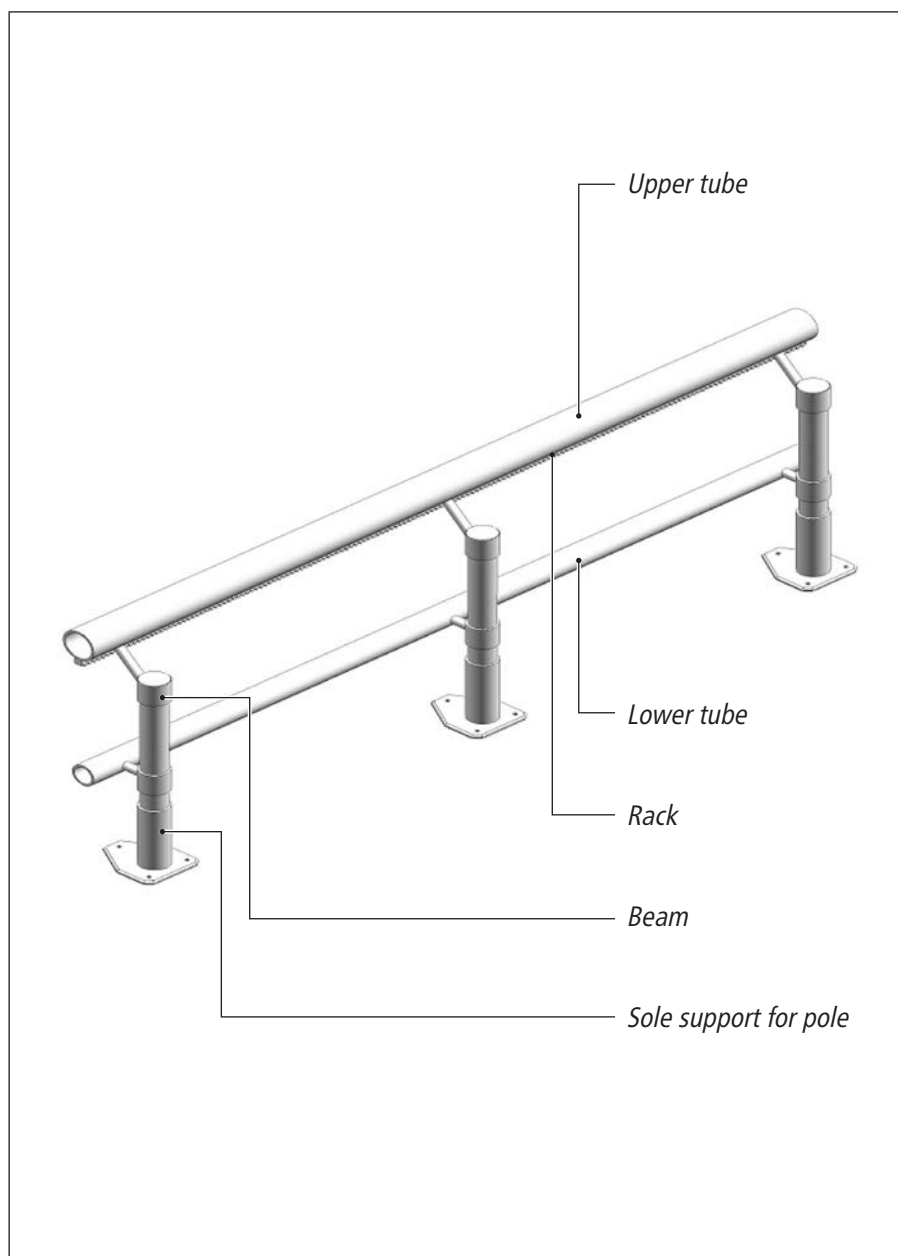
Fig. 1

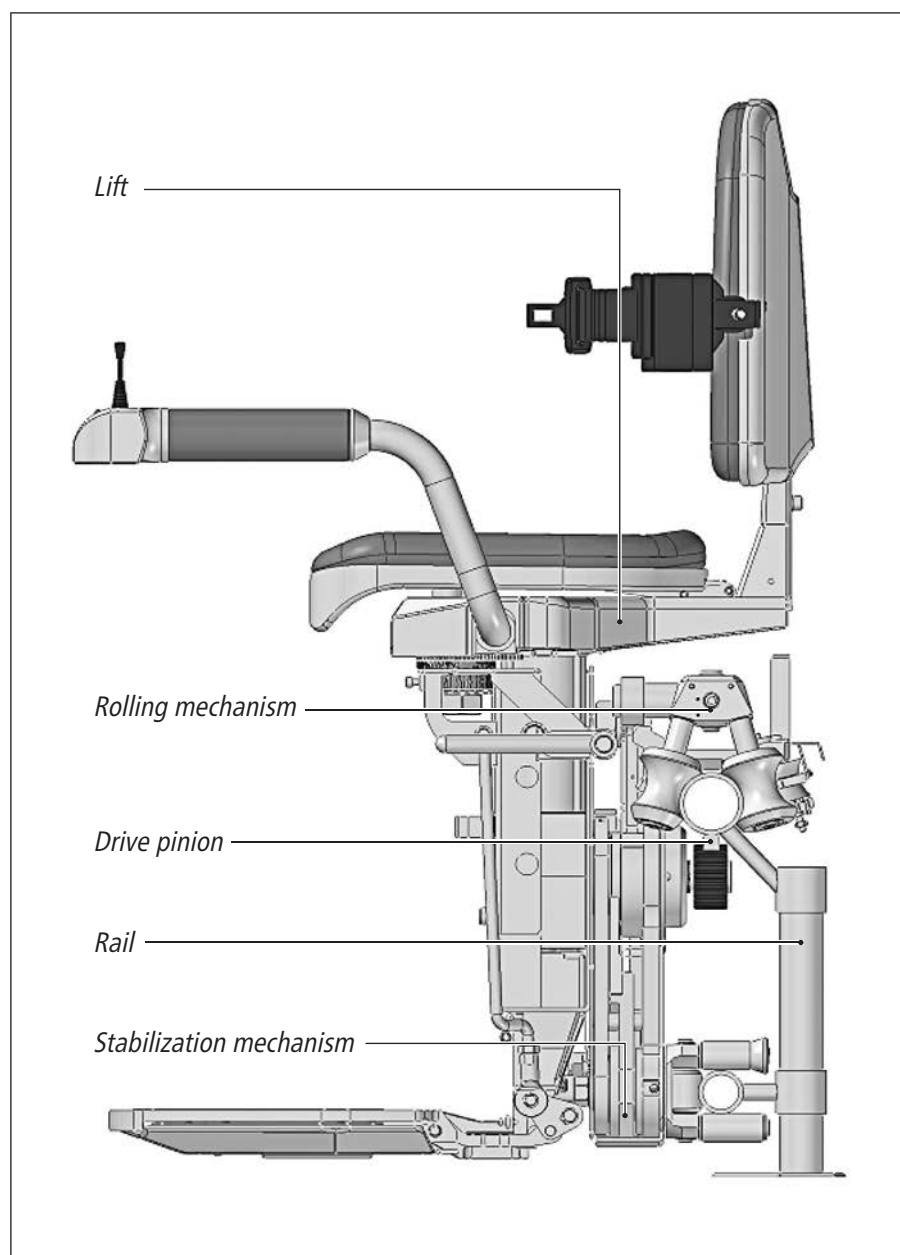
Fig. 2

3.2 TECHNICAL DATA

- Rated capacity – 125 kg
- Travel speed – $0,1 \text{ m/s} \pm 10\%$
- The inclination angle of the track measured against the horizontal:
- standard 0 - 58°

In exceptional cases, only following additional measures of protection and the written consent of the client, the lift can be used in the following conditions:

- ascent angle – max. 68°
- max. load 150 kg.
- The up time of the batteries without them being loaded is of approx. 10 trips, for a rail length of 6 m at an angle of ascent of 50°

The control joystick can be installed in the right arm or left arm.

3.3 ENVIRONMENTAL INFLUENCE

This model has no negative influence on the environment, produces no emissions, vibrations, noise, or anything else that could adversely impact the user.

The only thing to be observed is that, after changing the batteries (after a period of about 2-4 years of use), these should be taken to special locations in order to be recycled according to the regulations in force.

4. USE

4.1 HOW TO USE

Calling the lift from/to the parking area is made with the remote control in fig. 3, by pressing one of the two keys labeled with arrows.

Keep the button pressed without worrying because the lift will slow down and stop automatically when it reaches the end of the track where you are.

We do not recommended that you press + release the remote button when it reaches the end of the track because the lift will stop.

Fig. 3

Before using the remote, always make sure that no one is getting on or off the lift at that time. The remote control can also move the lift even if it is not in your line of sight.

To occupy less space, the lift can be folded in the parking area, as shown in fig. no. 4

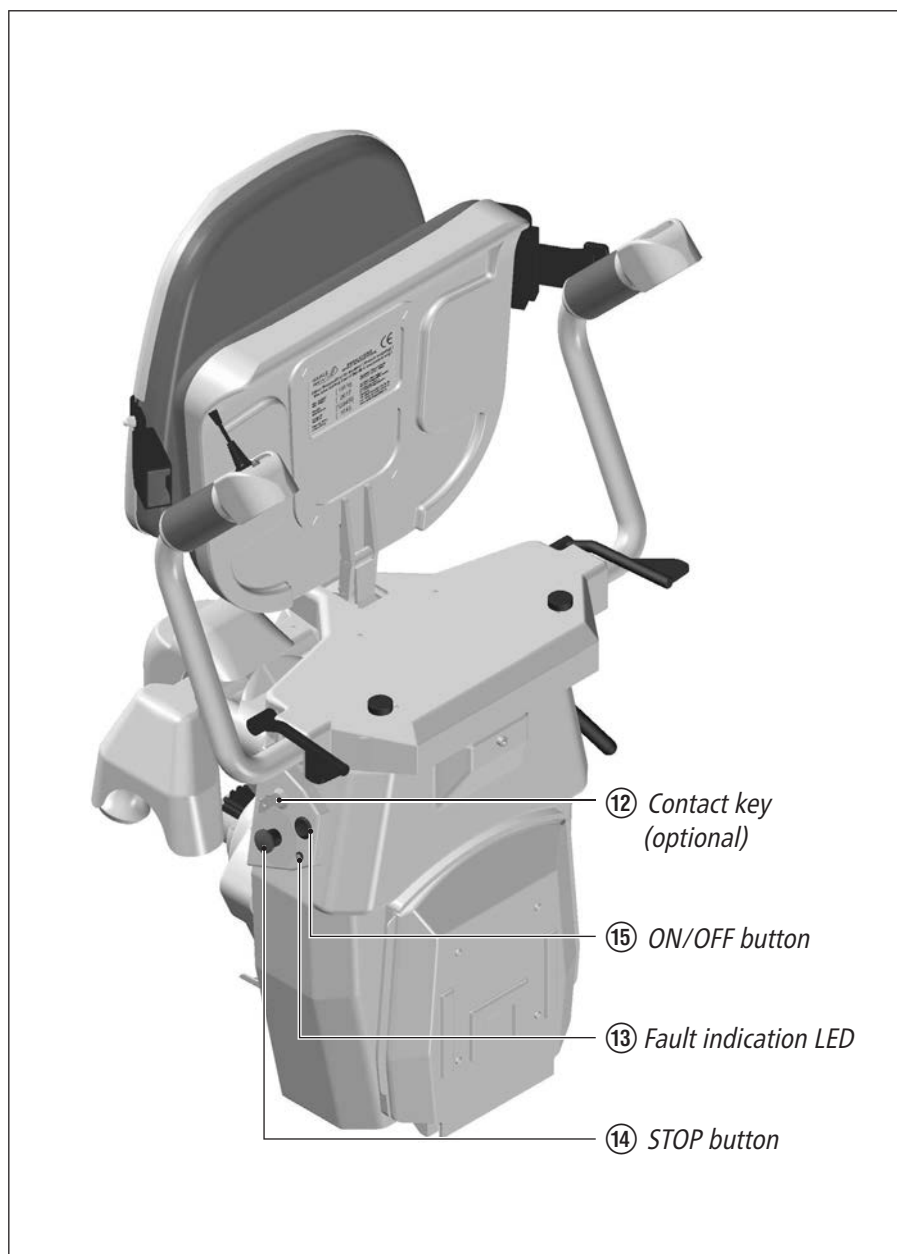
Fig. 4

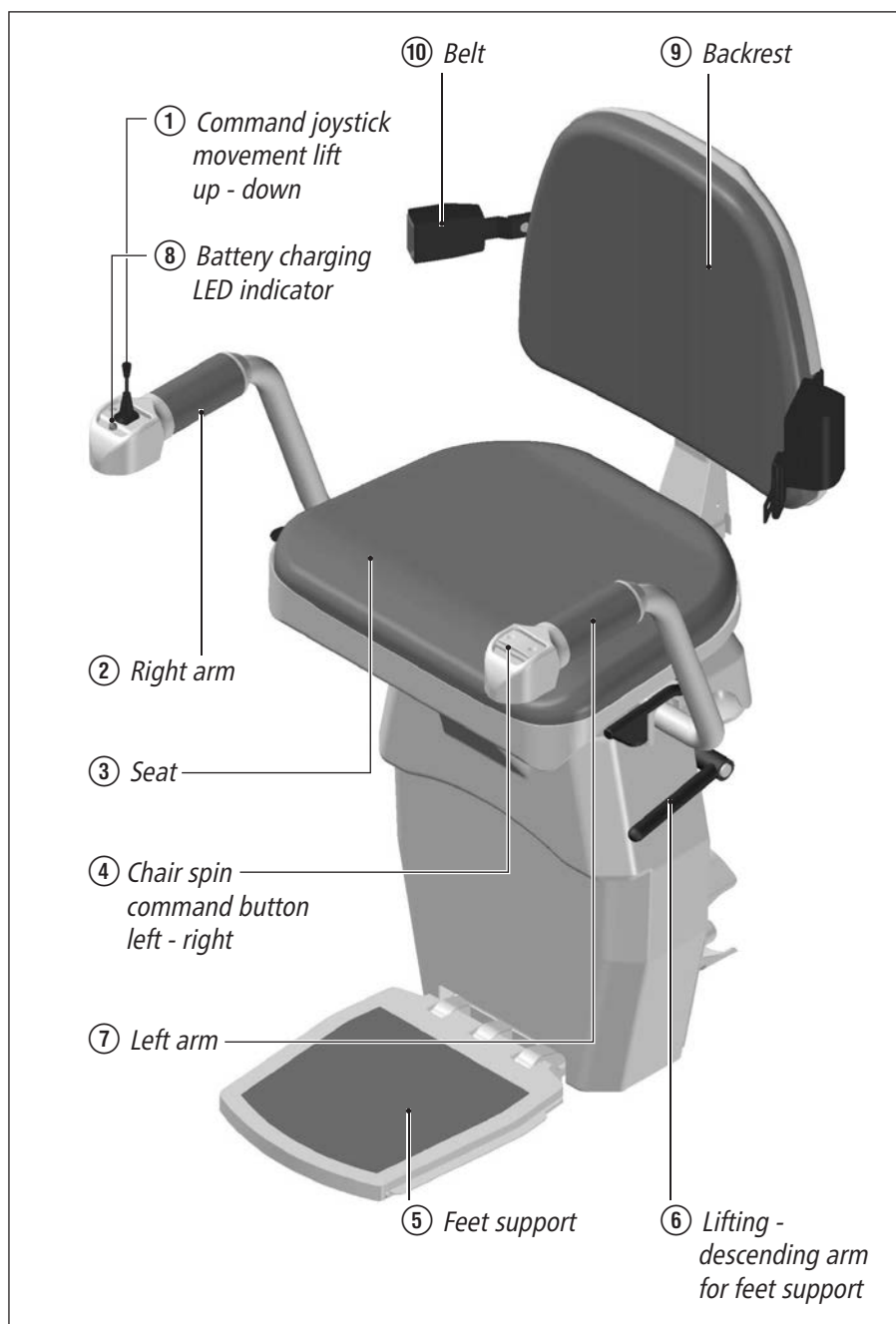
Fig. 5

To put the lift into operation, proceed as follows:

- bring down the arms ② and ⑦ from the vertical position, as shown in fig. 4, to a horizontal position, as shown in fig. 5.
- the same applies to the seat ③
- by lifting the handle ⑥, you can also tilt the foot rest ⑤ from a folded position fig. 4 to a use position fig. 5; after that, you can get on the lift. Starting and moving the lift is made by moving the joystick ① fig. 5, as follows:
- to move to the right, move the joystick to the right and hold.
- to move to the left, move the joystick to the left and hold
- we do not recommend that you interrupt the command unless in case of obstacles
- to stop the lift at either end of the track, the control joystick must remain actuated until the lift reaches the charging contacts and stops on its own, followed by the blue LED ⑧ on the joystick arm lighting up, this indicating that the battery is charging from the power supply
- if the lift is stopped on the track outside the charging elements, it will beep to warn that the battery is not charging, and therefore that it must be moved so that the lift may reach the charging contacts
- when the travel ends and the top of the track is reached, the seat ③ with backrest and arms ② and ⑦ fig. 5 are rotated to max. 90° to facilitate the user to get on and off the seat. The standard elevator has a fixed seat. Optionally it can be equipped with a manual rotation of the seat. Next are the steps for using this function:
- lift one of the handles ⑪ and slowly rotate the seat. After the seat starts to rotate release the handle ⑪ and continue the rotation in the direction you have to get of the lift. The rotation of the seat stops at 75° - fig no 5.1. if you want the seat to rotate to 90°, lift the handle ⑪ again and turn more in the same direction until the seat reaches 90°.
- to get back to the normal running position lift again the handle ⑪ and move in the opposite direction as described above until the seat reaches the normal running position.

Optionally, the stair lift can be equipped with electrical swivel seat function.

This movement is controlled by button ④ fig. 5, as follows:

- actuate the button in the right for right lifts or in the left for left lifts, and rotating will occur as long as the button is pressed down, automatically stopping at a max. angle of rotation of 90°
- to return the backrest and seat in normal travel position fig. 6, press the same button on the reverse side.

The lift does not move unless the seat is in the correct travel position.

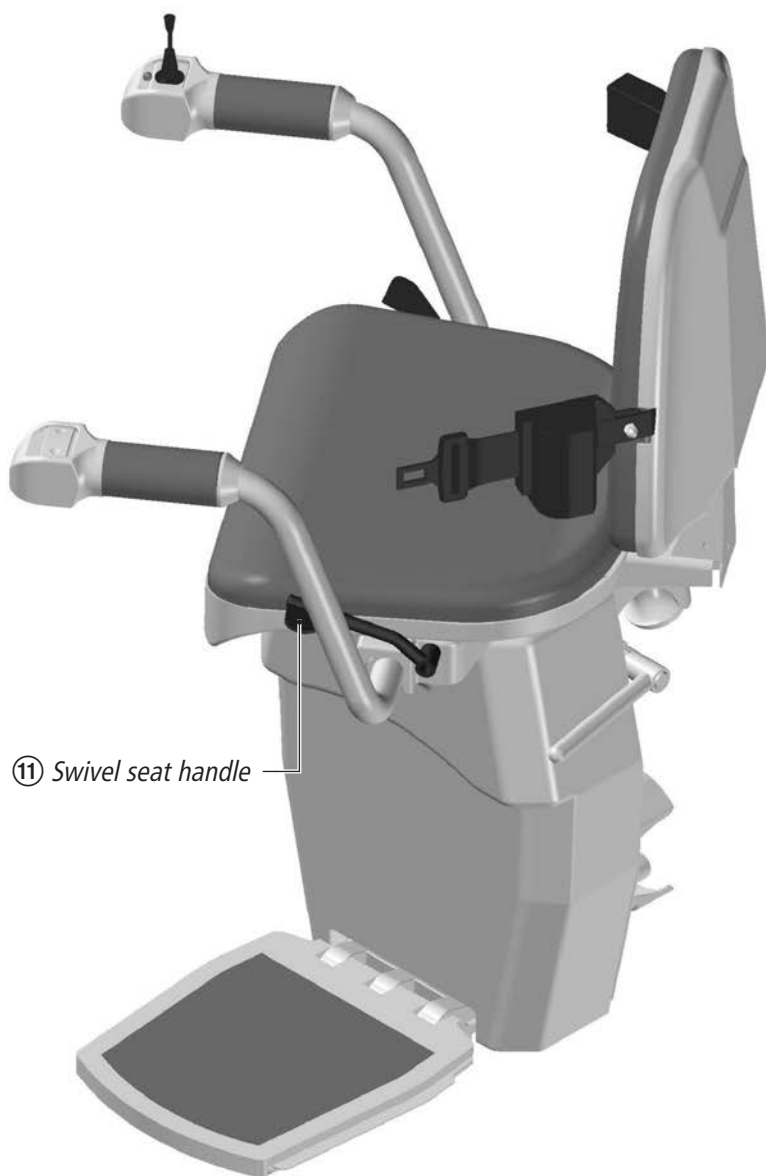
Fig. 5.1

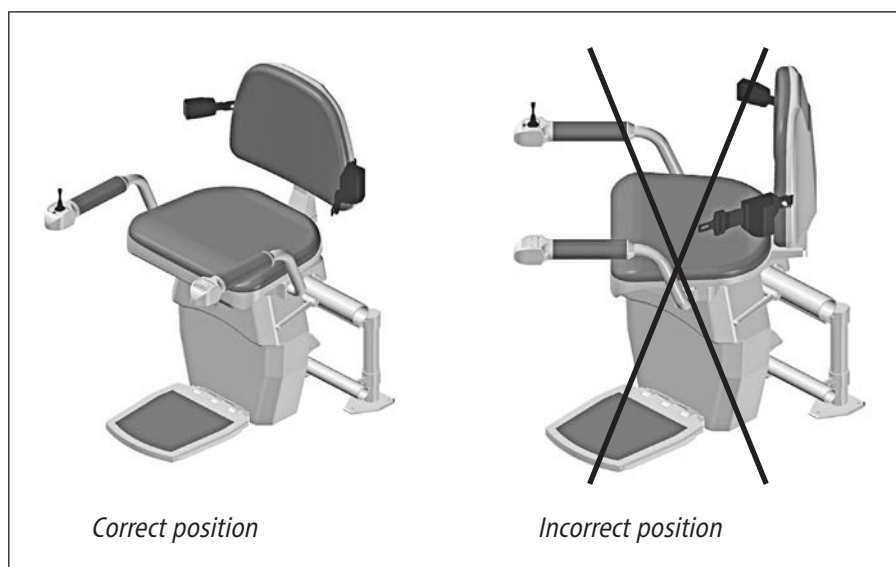
Fig. 6

- after the travel has ended, get off the lift and folds as follows:
- lift the seat ③, arms ② and ⑦, press the handle ⑥ fig. 5, then, using the remote, send the lift in the parking area

The LED ⑬ fig. 4 signals using a certain code (flashes) any possible malfunction. Communicating this code (how many times the LED flashes on and off) to the authorized lift maintenance company, interventions can be performed more quickly and effectively to remedy the situation.

In order to avoid unpleasant accidents, the lift is equipped with obstacle sensors for both track and stairs. When such a sensor actuates (when the lift hits an obstacle on the stairs or track), the travel of the lift in that direction stops, allowing the movement of the lift in the reverse direction to remove the obstacle. The lift areas provided with obstacle sensors are shown in fig. 7.

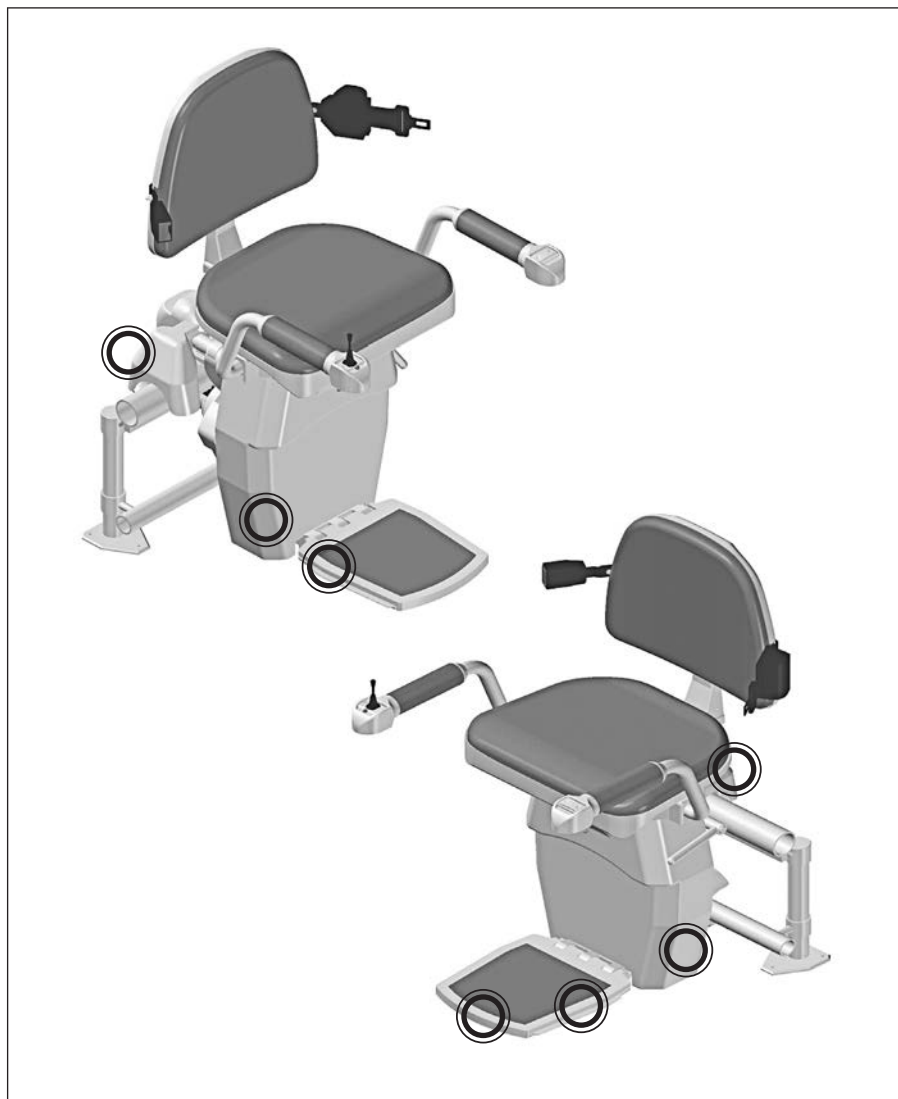
Fig. 7

The information regarding the characteristics of the elevator, the manufacturing date, the manufacturing series and the manufacturer can be found on the labels provided under the seat:

		
Member of HAWLE GROUP INTERNATIONAL		
Dieser Treppenlift ist für maximal 1 Person ausgelegt ! The save working load of this lift is one person only !		
Max. Gewicht Max. weight	125 kg	Hersteller / Manufacturer HAWLE H S.R.L., Biled
Baujahr Manufactured		Vertrieb / Distribution HAWLE Treppenlifte GmbH Am Bacherbusch 1 D - 53809 Ruppichteroth Germany
Serien-Nr. Serial-no.		Tel. 00 49 (0) 2295 / 90 93 40 Fax. 00 49 (0) 2295 / 90 93 41 www.hawle-treppenlifte.de
Farb-Nr. (RAL) Colour (RAL)		

4.2 PREVENTIVE MEASURES BEFORE USE

- check that both the seat and arms are used in the correct travel position according to fig. 6.
- before use, check that there are no obstacles on the rail or on the lift's travel area.
- use the lift only after removing such obstacles.
- make sure that no clothes are hanging that can be caught by the lift.
- keep your legs only on the foot support and do not stretch your legs over it.
- while moving, hold hands on the lift arms.
- always use the seat belt when you move using the stair lift
- sit correctly on the lift, do not lean back or forward.
- optionally, after use, remove the switch key and keep it in a safe place so as to prevent the lift from being used by children or other unauthorized persons.
- check that the battery charging source is not disconnected and that the battery LED shows the stair lift is charging

4.3 PROHIBITIONS

- traveling while standing on the lift
- using the lift in case of fire
- placing the lift in places with high humidity or where it may come in contact with water
- spraying it with liquid
- dirtying or lubricating the controls
- transport of flammable materials
- transport of objects
- exceeding the maximum load capacity
- washing or cleaning of the track

If the stair lift does not stop at the end of the rail either at the stainless steel reeds, or the limit switch or the L-bracket, the final limit switch will be actuated automatically. This switches off the lift. To deactivate the final limit switch, drive lift manually away from the charger using the supplied hand winder (see fig. 8).

Hand winder

Fig. 8

5. MAINTENANCE

5.1 BY THE USER

The only thing the user must do is to keep the lift and rail clean.

It is very important for the rail to be free of dirt because it can be pulled into the rollers, and, in time, it imprints in the paint, resulting in damage of the paint and of the rollers.

We recommend the following:

- after cca. 10 uses, but no more than a 3 days, clean the rail with a damp cloth and industrial alcohol or another non-corrosive solution
- if the lift was not used for a long period before starting, clean the rail and remove dust from the rollers
- washing or cleaning of the rack is prohibited, as it has been lubricated with grease at the time of the installation.

If you however have not complied with the above, and if the dust has not imprinted yet, the intervention of an authorized person for cleaning the plastic rollers is necessary.

MODEL HW11:

If the lift is not in use, it is absolutely necessary to protect the drive unit against weather effects with the included cover cap.

5.2 BY AUTHORIZED COMPANIES

No further interventions should be required during the warranty period, if using the lift correctly, and maintaining the appropriate cleanliness of the track and observing the annual revisions.

However, if irregularities occur, please contact the company that installed the lift.

After the warranty period, we recommend an annual check of the lift by a company authorized by the manufacturer.

5.3 MAINTENANCE LIST

5.3.1 WEAR PARTS

Wear parts for which we recommend checking are:

- all plastic guide and support rollers
- the sprocket
- other items that would show visible signs of wear.

5.3.2 SAFETY DEVICES

For safety devices and parts, check both their integrity and that of their seals. If any of this safety devices are not working correctly you may call the company that you purchased the stair lift from. Only authorized companies are allowed to inspect the safety devices.

ANNEX 1

PRODUCT INFORMATION

Type:
Series:
Date of purchase:

INSTALLER INFORMATION

Company name:
Contact person:
Headquarters:
Phone:
Fax:
E-mail:

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